

EXPERTS IN PATIENT HANDLING SOLUTIONS



X-RAY PLATE HOLDER – WASHABLE

This guide covers all products with the prefix XPH-Q in the product code.

X-ray Plate Holders allows health professionals to take mobile X-rays without having to directly manoeuvre patients.

USER INSTRUCTIONS



1

To use this product effectively you will need to use an **X-ray Insertion Tube** with the **X-ray Plate Holder**. This will help you to slide the cassette/imager behind your patient.



Insert the x-ray cassette/digital imager into the quilted **X-ray Plate Holder** so that when you insert the holder under the patient the black handles will be facing down.

2



3

Unroll an **X-ray Insertion Tube** under the patients torso, as shown here.

Once the **X-ray Insertion Tube** is unrolled, insert the quilted **X-ray Plate Holder** (with cassette/imager inside) into the tube and slide under the patient.

4



5

You can also use our **Extension Straps - Wipeclean** to help you, passing each to your colleague through the body's natural curves.

Once the x-ray has been taken, the cassette in its padded sleeve (**X-ray Plate Holder**) can be removed by simply sliding out on the opposite side to the side it was inserted.

6



MATERIAL CONTENT

Outer: 100% Polyester quilted
with Polyester Wadding
Handles: Polypropylene



STORAGE AND HANDLING

- Store in a clean dry environment that is not subject to excessive heat.
- Keep away from sunlight.
- Store flat and avoid placing heavy objects on top of the product.



Keep dry



Keep away from
sunlight

WARNINGS AND PRECAUTIONS


- Do not leave or drop on the floor; it is a slip hazard.
- Test for slippiness before use. If over-laundered and it doesn't slide please discard or replace.
- Inspect product (including handles) before use for signs of damage or tears.



Caution



Do not use if
package is damaged

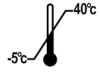

Storage Temperature
Limit

CLEANING INSTRUCTIONS

- Wash as per laundry instructions.
- Can be wiped clean, between use with the same patient, with natural water and detergent or wipes that don't contain alcohol, solvents, bleaching or abrasive agents.
- Cleaning materials used should be patient safe and biodegradable.



Wash at 30-75°C. Tumble dry on low heat. Do not Iron.
Do not use bleach. Do not use fabric conditioner.

DISPOSAL INSTRUCTIONS

- Non-recyclable
- Dispose of as clinical waste if it has been used by an infectious person.
- Otherwise dispose of through normal waste management.



Patient Information
website



Medical Device



MANUFACTURER

Hospital Direct (Marketing) Ltd, Units 2, 3 & 4, The Green,
Clun, Shropshire SY7 8LG, United Kingdom



EU REPRESENTATIVE: Obelis S.A., Bd Général Wahis 53,
1030 Brussels, Belgium



www.hospitaldirect.co.uk

HOSPITAL DIRECT (HD) ADDITIONAL SYMBOLS

At HD we understand that not everyone is familiar with all equipment, so to help therapists and healthcare professionals in assessing a product's suitability for a client, we have included two assessment tools, Functional Independence Measure (FIM) and Easy Guide Symbols. These will enable the healthcare professional to assess their client and decide whether or not a product will help the person based on their level of dependence and functional ability.

FUNCTIONAL INDEPENDENCE MEASURE (FIM) ASSESSMENT

Our FIM assessment guide for each product will help you decide the suitability of this product for the person's ability and need. Based on the standard criteria from Level 6 where the person can use the product unaided and unsupervised to Level 1 where all the assistance is provided by the carer and the client can do nothing, this guide is easy, quick and simple to use to check suitability against ability and circumstance.

FIM LEVELS QUICK GUIDE. SUBJECT VS CARER HELP



EASY GUIDE SYMBOLS

We use four symbols to indicate where a product is suitable and safe for the person to use unassisted, with a single carer or with multiple carers. These symbols indicate the minimum recommendation.



HD has a policy of continuous development and, as such, reserves the right to alter specifications (including measurements, materials and colours) without prior notice. If any serious incident has occurred in relation to the device it should be reported to the manufacturer and the competent authority in which the user and/or patient is established.