




LOCKING LOOPS user instructions

This guide covers the use of locking loops where present in SIBA Top Draw sheets.


If your SIBA Top Draw sheet has Locking Loops attached and you wish to use them to secure the sheet to the bed, please follow these instructions;

Lay the SIBA 4 Way Top Draw sheet over the top end of the bed, following the direction label so the arrow points to the top middle edge of the bed base.

The Locking Loops should hang over the top when the head section is raised.

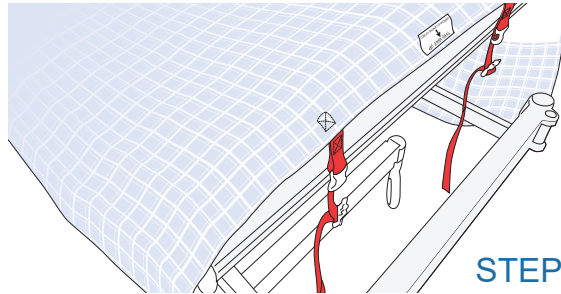


STEP 1

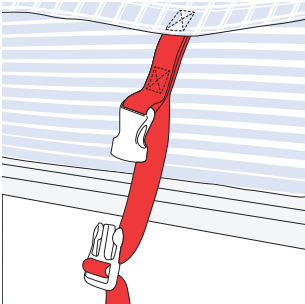

STEP 2


Pull downwards on the loose hanging end, allowing the strap to run through the male part of the clasp to extend the length of the strap.

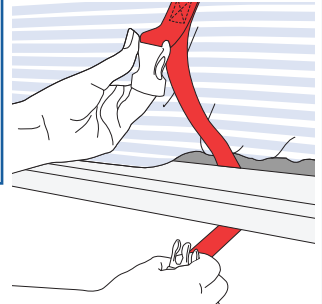

STEP 3

Feed the longer back strap through the top bar of the bed frame.

Ensure the part of the frame it is attached to is a part that moves when the back of the bed is raised.

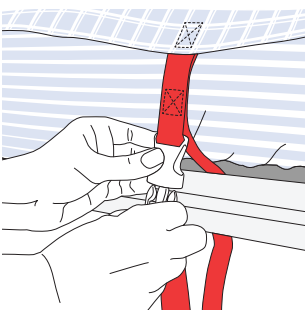


STEP 4

Buckle up!

Push firmly against the top strap with the palm of your left hand.

Pull down vertically the long loose end of the bottom strap. The strap will slide through the bottom of the clasp to tighten.

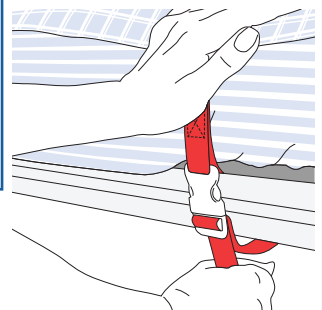

STEP 5