

EXPERTS IN PATIENT HANDLING SOLUTIONS




HDDELUXE™

HD DELUXE™ – POSITIONING SUPPORT TUBE

This guide covers the product code FST.

Our **HD DELUXE™ – Positioning Support Tube** is designed to provide ultimate lying comfort. With an extremely comfortable pressure reducing foam. Mouldable and very comfortable so users can benefit from the stability the tubes offer as well as knowing that their skin, integrity and pressure points are cared for. It can be evenly moulded into many shapes to cater for the specific needs of each user. Below are some examples.


USER INSTRUCTIONS

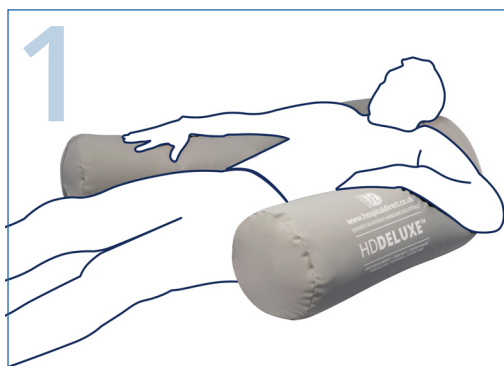

Comfortable supine lying supporting head, shoulders and elbows.




Comfortable side lying, supporting head, knees (shown with **HD Deluxe Limb Pad**), buttocks and lower limbs.

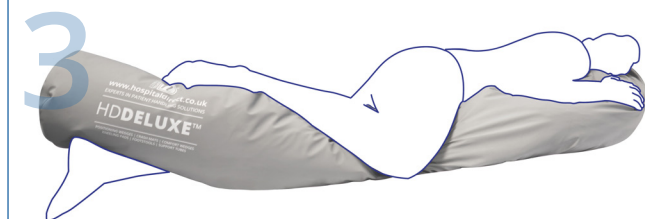

Full length body support taking pressure off knees, ankles and heels) and supporting ultimate lying comfort.

Full length body support offering a mouldable, comfortable and safe position, protecting head, arms, knees and heels.

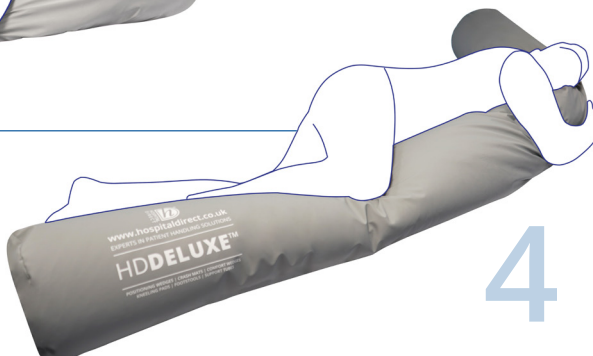

MATERIAL CONTENT

Cover Fabric: 100% polyester

Cover Coating: 90% PVC + 10% PU

Foam: Polyether Foam

Biocompatible cover (Oekotex certified) can be used directly on the skin



STORAGE AND HANDLING

- Store in a clean dry environment that is not subject to excessive heat.
- Keep away from sunlight.
- Store flat and avoid placing heavy objects on top of the product.

WARNINGS AND PRECAUTIONS

- Do not leave the on the floor when not being used.
- Do not use on a wet floor.
- Always test product grip on your chosen surface before using.
- Inspect before use for signs of wear or damage.

CLEANING INSTRUCTIONS

- Wipeclean with natural water and detergent or wipes that don't contain alcohol, solvents, bleaching or abrasive agents.
- Cleaning materials used should be patient safe and biodegradable.

DISPOSAL INSTRUCTIONS

- Non-recyclable.
- Dispose of as clinical waste if it has been used by an infectious person.
- Otherwise dispose of through normal waste management.



Keep dry




Keep away from sunlight



Caution



Do not use if package is damaged

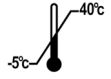

Storage Temperature Limit



Wipeclean Only



Patient Information website



Medical Device



MANUFACTURER

Hospital Direct (Marketing) Ltd, Units 2, 3 & 4, The Green, Clun, Shropshire SY7 8LG, United Kingdom



EU REPRESENTATIVE: Obelis S.A., Bd Général Wahis 53, 1030 Brussels, Belgium



www.hospitaldirect.co.uk

HOSPITAL DIRECT (HD) ADDITIONAL SYMBOLS

At HD we understand that not everyone is familiar with all equipment, so to help therapists and healthcare professionals in assessing a product's suitability for a client, we have included two assessment tools, Functional Independence Measure (FIM) and Easy Guide Symbols. These will enable the healthcare professional to assess their client and decide whether or not a product will help the person based on their level of dependence and functional ability.

FUNCTIONAL INDEPENDENCE MEASURE (FIM) ASSESSMENT

Our FIM assessment guide for each product will help you decide the suitability of this product for the person's ability and need. Based on the standard criteria from Level 6 where the person can use the product unaided and unsupervised to Level 1 where all the assistance is provided by the carer and the client can do nothing, this guide is easy, quick and simple to use to check suitability against ability and circumstance.

FIM LEVELS QUICK GUIDE. SUBJECT VS CARER HELP



EASY GUIDE SYMBOLS

We use four symbols to indicate where a product is suitable and safe for the person to use unassisted, with a single carer or with multiple carers. These symbols indicate the minimum recommendation.



Person Unassisted



Single Carer



Two Carers



Four Carers

HD has a policy of continuous development and, as such, reserves the right to alter specifications (including measurements, materials and colours) without prior notice. If any serious incident has occurred in relation to the device it should be reported to the manufacturer and the competent authority in which the user and/or patient is established.