



EXPERTS IN PATIENT HANDLING SOLUTIONS

HD **DELUXE**™

HD DELUXE™ – COMFORT LEG SUPPORT

This guide covers the product code CLT.

Our **HD Deluxe™ – Comfort Leg Support** is designed to provide your end users with the ultimate in lower leg support. Designed so that the heels can remain free hanging and so free of pressure, the support for the legs is provided by the internal chipped viscoelastic foam that allows each particle of memory foam to react directly to the user's leg shape and weight, creating a super soft surface to rest their legs on that is specific to them and that can mould to support their particular needs, providing pressure relief at all times. The outer cover is fully welded, waterproof, vapour permeable. They also reduces the risk of pressure sores developing as the particles are always moving.

USER INSTRUCTIONS



- ### 1 Positioning the Legs

Place the **Comfort Leg Support** on a stable, flat surface i.e. bed/mattress.
Using correct manual handling techniques, gently lift and position the user's legs into the channel sections of the **Comfort Leg Support**.
- ### 2 Check Heel Position

Ensure the heels are clear hanging over the **Comfort Leg Support**, resting just beyond the end for correct positioning.
- ### 3 Suitable for Different Leg Sizes

The **Comfort Leg Support** is suitable for all legs, including standard legs, plus size legs, and oedematous (swollen) legs.
Adjust positioning if needed to ensure comfort and support.
- ### 4 Final Comfort and Safety Check

Confirm the legs are well supported and aligned.
Recheck that the heels remain free of pressure.
Adjust if required before use.
- ### 5 Fitting the Washable Cover (Optional)

Stretch the elasticated cover over the **Comfort Leg Support** so it fits snugly.
The cover is a nice snug fit, however, if you believe a crease could be formed, you can smooth out creases for comfort either by hand, or using one of our **Slippy Sleeves (SS002)**.

MATERIAL CONTENT

Cover Fabric: 100% polyester
Cover Coating: Polyurethane
Chipped Viscoelastic Foam: Polyurethane

STORAGE AND HANDLING

- Store in a clean dry environment that is not subject to excessive heat.
- Keep away from sunlight.
- Store flat and avoid placing heavy objects on top of the product.

WARNINGS AND PRECAUTIONS

- Do not leave the on the floor when not being used.
- Do not use on a wet floor.
- Always test product grip on your chosen surface before using.
- Inspect before use for signs of wear or damage.

CLEANING INSTRUCTIONS

- Wipeclean with natural water and detergent or wipes that don't contain alcohol, solvents, bleaching or abrasive agents.
- Cleaning materials used should be patient safe and biodegradable.

DISPOSAL INSTRUCTIONS

- Non-recyclable.
- Dispose of as clinical waste if it has been used by an infectious person.
- Otherwise dispose of through normal waste management.



Keep dry



Keep away from sunlight



Caution



Do not use if package is damaged



Storage Temperature Limit



Wipeclean Only



Patient Information website



Medical Device



MANUFACTURER

Hospital Direct (Marketing) Ltd, Units 2, 3 & 4, The Green, Clun, Shropshire SY7 8LG, United Kingdom



EU REPRESENTATIVE: Obelis S.A., Bd Général Wahis 53, 1030 Brussels, Belgium



www.hospitaldirect.co.uk

HOSPITAL DIRECT (HD) ADDITIONAL SYMBOLS

At HD we understand that not everyone is familiar with all equipment, so to help therapists and healthcare professionals in assessing a product's suitability for a client, we have included two assessment tools, Functional Independence Measure (FIM) and Easy Guide Symbols. These will enable the healthcare professional to assess their client and decide whether or not a product will help the person based on their level of dependence and functional ability.

FUNCTIONAL INDEPENDENCE MEASURE (FIM) ASSESSMENT

Our FIM assessment guide for each product will help you decide the suitability of this product for the person's ability and need. Based on the standard criteria from Level 6 where the person can use the product unaided and unsupervised to Level 1 where all the assistance is provided by the carer and the client can do nothing, this guide is easy, quick and simple to use to check suitability against ability and circumstance.

FIM LEVELS QUICK GUIDE. SUBJECT VS CARER HELP



EASY GUIDE SYMBOLS

We use four symbols to indicate where a product is suitable and safe for the person to use unassisted, with a single carer or with multiple carers. These symbols indicate the minimum recommendation.



HD has a policy of continuous development and, as such, reserves the right to alter specifications (including measurements, materials and colours) without prior notice. If any serious incident has occurred in relation to the device it should be reported to the manufacturer and the competent authority in which the user and/or patient is established.