

REUSABLE

EXPERTS IN PATIENT HANDLING SOLUTIONS



EcoMove™ Reusable Transfer Sheet

This guide covers all products with the prefix RTSH in the product code.

The **EcoMove™ Reusable Transfer Sheet** enables users to transfer their patients from one lying surface to another across a lateral transfer board. Available with 5 handles on each side.



USER INSTRUCTIONS



- 1 Fold the **EcoMove™ Reusable Transfer Sheet** in half and insert under the log rolled patient.

Pull the sheet out so it lies flat and covers half the lying surface.

2



3

- Log roll patient in other direction and pull out the rest of the sheet so it is flat.

The patient should now be lying fully on the sheet and all handles should be accessible.

4



5

- Bring the two surfaces together. Then staff on the furthest side of the patient can now use the handles of the sheet to log roll the patient towards them to allow the lateral transfer board (lying on the receiving surface) to be inserted easily.

Lower the patient gently, slide the feet onto the board and, using co-ordinated commands, slide the patient gently from the giving surface to the receiving surface. It can help if the giving surface is slightly higher than the receiving surface.

6



7

Once the patient is safely on the receiving surface, remove the board by log rolling the patient.



Reposition patient.

8



9

The patient can remain on the sheet until it is no longer needed.



10

Remove the sheet after use.



MATERIAL CONTENT

Fabric: 100% polyester with 50% recycled content

Handles: Woven cotton



STORAGE AND HANDLING

- Store in a clean dry environment that is not subject to excessive heat.
- Keep away from sunlight.
- Store flat and avoid placing heavy objects on top of the product.

WARNINGS AND PRECAUTIONS

- Do not leave the on the floor when not being used.
- Inspect the product, including handles, before use for signs of damage or tears.
- Safe Working Load (SWL): 500 kg

CLEANING INSTRUCTIONS

- Wash as per laundry instructions.
- Can be wiped clean, between use with the same patient, with natural water and detergent or wipes that don't contain alcohol, solvents, bleaching or abrasive agents.
- Cleaning materials used should be patient safe and biodegradable..

DISPOSAL INSTRUCTIONS

- Recyclable-suitable for mechanical or chemical polyester recycling. Where possible, repair and reuse.
- Recycle through your nearest recycling point; fabric is not designed for kerbside collection.
- If the fabric is used in circumstances where they have come into contact with an infectious person or environment, dispose of it as clinical waste.



Caution



Keep dry



Keep away from
sunlight



Do not use if
package is damaged



Storage Temperature
Limit



Wash at up to 90°C. Tumble dry on low heat. Do not iron.
Do not use bleach. Do not use fabric conditioner.



Patient information
website



Medical Device



MANUFACTURER

Hospital Direct (Marketing) Ltd, Units 2, 3 & 4, The Green, Clun, Shropshire SY7 8LG, United Kingdom

EU REPRESENTATIVE: Obelis S.A., Bd Général Wahis 53, 1030 Brussels, Belgium

EU REP



www.hospitaldirect.co.uk

HOSPITAL DIRECT (HD) ADDITIONAL SYMBOLS

At HD we understand that not everyone is familiar with all equipment, so to help therapists and healthcare professionals in assessing a product's suitability for a client, we have included two assessment tools, Functional Independence Measure (FIM) and Easy Guide Symbols. These will enable the healthcare professional to assess their client and decide whether or not a product will help the person based on their level of dependence and functional ability.

FUNCTIONAL INDEPENDENCE MEASURE (FIM) ASSESSMENT

Our FIM assessment guide for each product will help you decide the suitability of this product for the person's ability and need. Based on the standard criteria from Level 6 where the person can use the product unaided and unsupervised to Level 1 where all the assistance is provided by the carer and the client can do nothing, this guide is easy, quick and simple to use to check suitability against ability and circumstance.

FIM LEVELS QUICK GUIDE. SUBJECT VS CARER HELP



©2018 Hospital Direct (Marketing) Ltd

EASY GUIDE SYMBOLS

We use four symbols to indicate where a product is suitable and safe for the person to use unassisted, with a single carer or with multiple carers. These symbols indicate the minimum recommendation.



Person Unas-



Sin-
gle



Two
Car-



Four
Car-

HD has a policy of continuous development and, as such, reserves the right to alter specifications (including measurements, materials and colours) without prior notice. If any serious incident has occurred in relation to the device it should be reported to the manufacturer and the competent authority in which the user and/or patient is established.